


Mobile Mounting Instructions

Overview

This document explains how to correctly install the Virtua ECO Docking Tray (Cat. # V-ECO-TRAY) and use it to mount a Codonics Virtua® ECO Series Medical Disc Publisher in a mobile environment.

Locating and Drilling Mounting Holes

1. Place the Virtua ECO Docking Tray on the countertop where it will be mounted.
2. Make sure that there are 4 in. (102 mm) of clearance between the rear and side tray walls, then use the tray hole pattern to mark the locations of the four mounting holes.

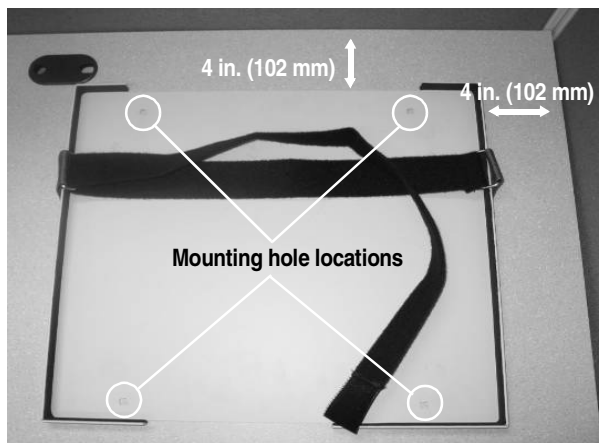

3. Move the Docking Tray aside and drill holes 3/8 in. (8 mm) in diameter at each of the locations marked above.

NOTE: Owners of Horizon® or Horizon Cabinet can mount the Virtua ECO Docking Tray in its place. To accommodate the square neck of the carriage screws, you must drill the holes wider with a 3/8-in. (8-mm) drill bit. After widening the holes, go to "Installing the Virtua ECO Docking Tray."

Installing the Virtua ECO Docking Tray

1. Align the Docking Tray square mounting holes to the previously drilled countertop holes.

2. Place one 2-in. (51-mm) carriage screw down through each Docking Tray mounting hole, and secure the tray by placing one 0.040-in. (1-mm) thick washer and one lock nut on each screw from underneath the countertop.
3. While holding down the head of the carriage screw, tighten the lock nut into the washer using a 7/16-in. (11-mm) socket and driver, a 7/16-in. (11-mm) crescent wrench, or an adjustable wrench.

NOTE: If you are installing the Virtua ECO Docking Tray to the Horizon Cabinet, you may require a deep socket driver to sufficiently tighten the lock nuts.

4. If desired, cover the extra length of the carriage screws with the included plastic screw caps.

If you have any difficulty installing your Virtua ECO Docking Tray, contact Codonics.

Mounting Virtua ECO in the Docking Tray

1. Place the Controller into the Docking Tray. Slide the Controller all the way forward until the front skin touches the inside of the tray and the Controller is centered between the side walls.

WARNING: Be careful not to pinch fingers between the Controller and Docking Tray walls.

2. Place the Recorder onto the Controller as instructed in the Virtua ECO Reference Guide under "Assembly."
3. Pull the Velcro strap snugly around the Recorder and fasten it as shown. Do not overtighten the Velcro strap.



Get it all with just one call
1-800-444-1198

All registered and unregistered trademarks are the property of their respective owners. Specifications subject to change without notice. Patents pending.

Copyright © 2008 Codonics, Inc. Printed in the U.S.A. Part No. 901-378-001.01.



17991 Englewood Drive
Middleburg Heights, OH 44130 USA
(440) 243-1198
(440) 243-1334 Fax
Email info@codonics.com
www.codonics.com

Codonics Limited KK
New Shibaura Bldg. F1
1-3-11, Shibaura
Minato-ku, Tokyo, 105-0023 JAPAN
Phone: 81-3-5730-2297
Fax: 81-3-5730-2295