

# Codonics Virtua eFilm Lite Viewer

**Technical Brief**

## Overview

The Codonics Virtua® Medical Disc Publisher provides support for eFilm® Lite from Merge Healthcare as an optional DICOM viewer.

eFilm Lite requires a Feature Key on Virtua to activate.

## Minimum System Requirements

eFilm Lite requires the following minimum system requirements on the host computer:

- ◆ Windows 7\* (Professional, Ultimate or Home Editions) or Windows Vista or Windows XP (Professional Edition)
- ◆ Intel® Pentium® II Processor
- ◆ 256 Mbyte RAM
- ◆ 800 x 600 pixel graphics (minimum display resolution)

\* Windows 7 support with Version 3.4 and higher.

## eFilm Lite Viewer Summary

eFilm Lite is a comprehensive viewing application which can be recorded on a CD or DVD, enabling the distribution of clinical studies. With eFilm Lite you can easily share clinical images with referring physicians or export images for marketing and educational purposes. Simply select the patient's study and record the study to a disc with Virtua, and the study is ready to be viewed on any PC.

The eFilm Lite Viewer is a more basic version of the popular eFilm Workstation® DICOM viewer, which is an application used for viewing and manipulating medical images. Digital images and data from various sources (including CT, MR, US, RF units, computed and direct radiographic devices, secondary capture devices, scanners, imaging gateways, or other imaging sources) can be displayed on workstations and laptops using this software. When viewing images, users can perform adjustments of window width and level, image stacking, annotation and measurement of regions of interest, and other image alterations.

## eFilm Viewer Recall

In October 2016, Codonics was notified by Merge Healthcare Incorporated of a recall of their eFilm Lite viewer. An updated version of the eFilm Lite viewer, version 4.1.1, that corrects the issue that triggered the recall, is available for download from Codonics.

**NOTE:** Virtua software version 5.0.0 and older contains the outdated version of the eFilm Lite viewer. If you have version 5.0.0 software or older, you must manually update your viewer. Refer to Codonics Virtua Viewer Installation Technical Brief 901-163-001 for details or contact our Technical support team.

*Get it all with just one call*  
**800.444.1198**

All registered and unregistered trademarks are the property of their respective owners. Specifications subject to change without notice.

Copyright © 2016 by Codonics, Inc. Printed in the U.S.A. Part No. 901-154-004.02.



17991 Englewood Drive  
Middleburg Heights, OH  
44130 USA  
+1.440.243.1198  
+1.440.243.1334 Fax  
Email info@codonics.com  
www.codonics.com

Codonics Trading Co, Ltd.  
317 Xianxia Rd. Building B  
Unit 1412  
Changning Dist., Shanghai  
P.R. China, 200051  
86-21-62787701  
86-21-62787719 Fax

Codonics Limited KK  
AQUACITY 9F,  
4-16-23, Shibaura  
Minato-ku, Tokyo,  
108-0023 JAPAN  
81-3-5730-2297  
81-3-5730-2295 Fax